AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

CaligaLoc is a medical device. It is an orthosis for stabilization of the upper and lower ankle after serious injuries.

Indications

- Non-surgical treatment of torn ankle ligaments or a badly sprained ankle joint (distortion)
- Protection following reconstructive ligament surgery
- Permanent stabilization for complex instability of the upper and / or lower ankle

⚠ Risks of using this product

- CaligaLoc must only be worn in accordance with the specifications contained in these instructions for use and the areas of application (places of use) listed. To ensure the optimum fit of the CaligaLoc, it is essential to check whether the product is adequately shaped. If necessary, the CaligaLoc can be anatomically re-shaped by a specialist.
- The use and duration of use must be discussed with the physician who prescribed the CaligaLoc. They will also inform you whether the CaligaLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the CaligaLoc as well as the instructions for use.

- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the CaligaLoc – e.g. an increase in discomfort – stop using it and contact your physician without delay.

Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation and movement disorders of the legs / feet, e.g. in the case of diabetes mellitus
- Impaired lymph drainage, including soft tissue swellings of uncertain origin located remotely from the fitted orthosis

Application instructions

Putting on / fitting the CaligaLoc

A – plastic shell, B – upper strap, C – lower strap, D – middle Y-shaped strap

We recommend that hosiery should be worn under the CaligaLoc.

Sit down on a chair to put on the CaligaLoc. Place your foot on the floor with your foot at a 90° angle to your lower leg, as this is the best position for fitting the orthosis.

- 1 Undo all the straps. Place the relevant foot on the sole of the CaligaLoc, taking care to ensure the correct position of the heel. The outside of the back of the foot should now be encased by the plastic shell.
 - 2 Now wrap the upper strap round your lower leg from behind and guide it through the loop in the shin bone area.
 - 3 Secure the end of the strap by fastening the Velcro to the section of strap which is underneath it.
 - 4 Guide the lower strap over the instep of the foot to the outside of the foot and secure it in place with the Velcro. If necessary, the strap can be cut with scissors to make it shorter.
 - 5 For the Y-shaped strap, proceed as follows: Guide the front strap in front of the shin bone and the rear strap round the heel into the guide loops, then use the Velcro to secure them to the section of strap underneath. Tighten the straps again if necessary.
- CaligaLoc is suitable for use in comfortable outdoor and sports shoes, and can also be worn without shoes overnight.

Taking off / removing the CaligaLoc

Undo all the Velcro straps and lift your foot up and forward out of the orthosis.

Cleaning instructions

Note: Never expose the CaligaLoc to direct heat (e.g. heater, sunlight, in your car, etc.! This can impair the effectiveness of the CaligaLoc.

Please refer to the cleaning instructions on the sewn-in label, which is located underneath the upper strap of the orthosis.

Wash the product by hand. Use a damp cloth to wipe the shell and cushion. Use a mild cleanser.

To make a complaint, please contact the retailer that provided you with the product.

Note: Only clean goods are handled.

Permissible conditions and places of use

In accordance with the indications (ankle joint).

Maintenance instructions

With correct handling and care, the product is practically maintenance-free.

Assembly instructions

The CaligaLoc is supplied ready for use.

Technical data / parameters

The CaligaLoc is an ankle orthosis with three attached straps. The straps are not removable.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the CaligaLoc have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polypropylene (PP), Polyamide (PA), Ethylene Vinyl Acetate (EVA), Steel, galvanized (Fe (Zn)), Silver-colored brass, nickel-free (CuSnZn), Polyester (PES), Acrylate

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2020-11

Specialists

General notes

- Check the optimal and individual fit of the CaligaLoc at the first fitting. Make any corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect the medical and therapeutic effectiveness of the product, thereby voiding any product liability.

Adapting the CaligaLoc

Adjusting the plastic shell

- 1 Check the fit of the CaligaLoc on the patient. Pay careful attention to the patient's individual anatomy (e.g. the lateral malleolus).
- 2 Carefully remove the cushion from the plastic shell.
- 3 If necessary, adapt the CaligaLoc using a hot-air gun at 120–140 °C (250–280 °F).
- 4 Wear protective gloves when you are doing this.
- 5 Glue the inner cushion back into the plastic shell.

Adjusting the cushion

- 6 If the ankle is badly swollen and the cushion is therefore pressing against the lateral malleolus, we recommend making a cutout in this part of the cushion. Use sharp scissors or a cutter knife and cut along the embossing.

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