AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Aluminium (Al Mg 3), Polyamid (PA), Polyurethan (PUR), Polyether, Polyoxymethylen (POM), Polyester (PES)

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ManuLoc / ManuLoc long are medical products. ManuLoc is an orthosis¹ for immobilizing the wrist. ManuLoc long is an orthosis¹ for immobilizing the wrist and parts of the forearm.

For optimum care, the ManuLoc is available in two styles:

- A ManuLoc: for stabilizing the wrist
- B ManuLoc long: for increased stabilization of the wrist

If the product is not used properly, all warranty rights will be rendered null and void.

Indications / Areas of Application

ManuLoc

- Post-traumatic or post-operative irritation
- Pre-operative care
- Carpal tunnel syndrome (mild)
- Tenosynovitis (moderate)
- Sprains / contusions
- Osteoarthritis (moderate)

ManuLoc long

- Post-traumatic or post-operative irritation, e.g. following treatment with a plaster cast in the case of radius fractures
- Pre-operative care
- Carpal tunnel syndrome (moderate)
- Tenosynovitis (moderate, with pain while resting and at night)
- Sprains / contusions
- Osteoarthritis (moderate)

Risks of using this product

Caution¹

- ManuLoc / ManuLoc long must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application (intended use, parts of the body) listed.
- To ensure the optimum fit of the ManuLoc / ManuLoc long, the size of ManuLoc / ManuLoc long required must be determined by a trained specialist².
- The sewn-in label of the ManuLoc / ManuLoc long, which contains information on the size, manufacturer, washing instructions and CE mark, is located in the insertion opening for the wide stay located on the inside of the hand (the volar stay). Only a trained specialist may carry out the (initial) fitting of the orthosis and provide instruction.
- Use in combination with other products, e.g. as part of compression treatment, must be discussed beforehand with your physician.
- Should you experience increased discomfort or notice any unusual changes, please contact your physician without delay.

- No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- This product is intended for the treatment of one patient only.
- No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed. Any medical aids – such as supports and orthoses³ – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, constrict the underlying blood vessels or nerves.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation and circulatory disorders of the arm / hand, e.g. in the case of diabetes mellitus.
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Putting on the ManuLoc / ManuLoc long

- 1 Start by loosening the three Velcro straps on the lower arm splint (or the four Velcro straps on the long version).
- 2 Place the ManuLoc / ManuLoc long on the correct hand. The straps are thus located on the top of the lower arm and on the back of the hand.
- 3 Fasten the straps on the lower arm splint in the following sequence: Lower arm straps 1, 2 and the strap over the back of the hand (or lower arm straps 1, 2, 3 and the strap over the back of the hand on the long version).

Taking off the ManuLoc / ManuLoc long

Loosen the straps on the lower arm splint in the reverse order. Take off the ManuLoc / ManuLoc long.

Cleaning instructions

Never expose the ManuLoc / ManuLoc long to direct heat (e.g. heaters, sunlight, in your car). This may damage the material, which can impair the effectiveness.

Hand wash the product at 30 °C (we recommend you use liquid detergent). Please note the following:

- Remove the Velcro straps if possible, or fasten them securely in position.
- Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.

We recommend that you use a mesh bag (spin cycle: maximum 500 rpm). Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.

This product has been tested by our in-house quality management system. If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Part of the body this product is used for

In accordance with the indications (wrist, lower arm). See Intended use.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The ManuLoc / ManuLoc long is supplied in an anatomically contoured state. The orthosis is equally suitable for both the right and left hand.

Technical specifications / parameters, accessories

The ManuLoc / ManuLoc long is a stabilizing orthosis for the wrist. It consists of a long lower arm splint.

The straps on the lower arm splint are detachable.

Spare part ManuLoc long – spare straps (x4), color: titan 129 5343 0080 000

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ManuLoc / ManuLoc long have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Aluminium (Al Mg 3), Polyamide (PA), Polyurethane (PUR), Polyether, Polyoxymethylene (POM), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2020-01

Specialists

General instructions

- Check the optimal and individual fit of the ManuLoc / ManuLoc long at the first fitting, and adjust if necessary.
- Practice the correct procedure for putting on the support with the patient.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect the medical effectiveness of the product, so that product liability is excluded.
- The ManuLoc / ManuLoc long is intended for the treatment of one patient only.

The aluminum stays are anatomically contoured, but they can still be individually adjusted to the patient.

Shaping the stays:

- 1 Carefully remove the stays from the insertion pockets and shape them to the anatomy of the patient. Reinsert the three stays into the pockets provided.

¹ Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product)

² A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

³ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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