Liebe Patientin, lieber Patient,

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Dear Patient,

thank you for placing your trust in Bauerfeind. By purchasing the VenoTrain glider donning aid, you have selected a product that meets very high medical and quality standards. The excellent workmanship provides you with the convenience you require to put on your compression stockings quickly and accurately, day after day.

Please read through all of these instructions for use carefully as they contain important information about how to use your donning aid. Please keep the instructions for use safe, as you may want to read them again later. If you have any questions, please contact your physician or medical retailer.

Intended purpose

The VenoTrain glider is an accessory to a medical device. It makes it easier to put on medical compression stockings for the lower and upper extremities.

Indications

- Restricted mobility

Application Instructions

Donning aid (legs)

- Put the foam surface – point forward – on the floor in front of you.
- Place the VenoTrain glider on top, with the white side facing upward. Make sure that the blue loop protrudes slightly beyond the tip of the foam surface.
- Place your toes in the marked semicircle.
- then push your foot gently forward with slight pressure. Your foot is now in the donning aid.
- Put on your compression stocking as usual until you reach your knee. Make sure that the blue tape sticks out from the stocking.
- a) For compression stockings with closed toe:
Grip the blue tape and pull the VenoTrain glider slowly and steadily out of the compression stocking while gently moving your toes.

b) For compression stockings with open toe:
Pull the VenoTrain glider out of the stocking with the loop at the front.

7 Bring your compression stocking into its final position, depending on the respective stocking style.

Important: After use, turn the VenoTrain glider to its correct side (with the loop pointing outwards).

Removing aid (leg)

- Put the pull tape in the VenoTrain glider and slide in your foot. The blue side of the donning aid is up.
- Fold up your compression stocking to the VenoTrain glider and place the edge or stay-up tape over it.
- Pull the compression stocking over the heel and instep towards the tip of your foot. When you remove the stocking, your VenoTrain glider is inside it.

Donning aid (arm)

- Put the pull tape in the VenoTrain glider and slide your arm in it. The blue side of the donning aid is up.
- You can don your compression stocking as usual starting with the hand over your VenoTrain glider to the upper arm. Adjust your stocking on the arm.
- Attach the loop of the VenoTrain glider at a safe point and simply pull it out while you hold the compression stocking with your free arm.

Cleaning Instructions

The VenoTrain glider is hard-wearing and absolutely easy to clean. Just wipe it with a moist cloth – that is all. Folded up and tucked away in the storage bag, the very light-weight VenoTrain glider fits any carrying bag. The way how to use the VenoTrain glider is illustrated on the following pages.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optically can help you. Failure to follow the instructions may also restrict the warranty.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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