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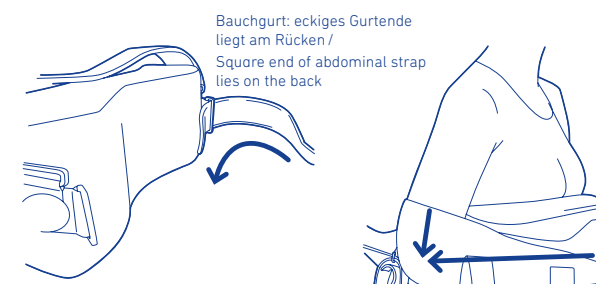
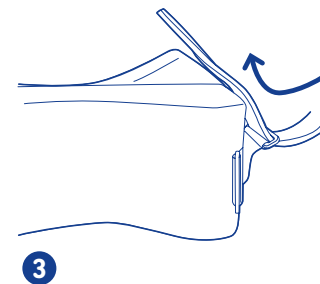
Polyurethan (PUR), Polyamid (PA), Polyoxymethylen (POM), Elastan (EL), Polyester (PES), Polypropylen (PP), synthetische Harze, Styren-Ethylen / Butylen-Styren-Blockcopolymer (TPS-SEBS)

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

## Intended purpose

OmoLoc 15 is a medical device. It is an orthosis<sup>1</sup> for immobilization and relief of the shoulder.

## Indications

All indications that require immobilization and relief of the shoulder joint, e.g.:

- Pre-operative care
- Post-operative treatment
- Post-traumatic treatment
- Non-surgical care

### Treatment before and after surgery:

- Fractures
- Shoulder replacement surgery
- Stabilization surgery (e.g. labrum refixation, glenoid augmentation)
- Tendon reconstruction (e.g. rotator cuff tears, superior capsular reconstruction (SCR), tendon transfer, tenodesis of the biceps tendon)
- Ligament injuries (e.g. separated shoulder)

### Non-surgical treatment for:

- Acute / chronic bursitis
- First anterior shoulder dislocation
- Impingement syndrome

## ⚠ Risks of using this product

- Please make sure that the complete forearm can be supported in the arm pocket. This is the only way to ensure effective treatment.
- OmoLoc 15 must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application (intended use, parts of the body) listed.
- To ensure the optimum fit of the OmoLoc 15, the length of the straps must be adjusted by a trained specialist<sup>2</sup>.
- The sewn-in label of the OmoLoc 15, which contains information on the size, manufacturer, washing instructions and CE mark, is located in the arm pocket, in the elbow area.
- Only a trained specialist may carry out the (initial) fitting of the orthosis and provide instruction.
- The use and duration of use must be discussed with the physician who prescribed the product. They will also inform you whether the orthosis can be used simultaneously with other products, and if so, which ones.
- Please note all the information you received with the product as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- Avoid contact with ointments, lotions, or substances containing grease or acids and keep the product away from direct sunlight.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the product – e.g. an increase in discomfort – stop using it and contact your physician without delay.

## Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation and circulatory disorders of the arms (e.g. diabetes mellitus)
- Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion)
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid
- Mental and physical disabilities that render it impossible to use the orthosis safely

## Application instructions

### Instructions for specialists

- A – Shoulder abduction cushion
- B – Forearm pocket

- C – 2 × forearm Velcro strip
- D – Shoulder strap cushion
- E – Abdominal strap and shoulder strap including D (shoulder strap cushion)
- F – 2 × buckle

OmoLoc 15 must be adapted by a trained specialist<sup>2</sup> before the first fitting. The shoulder strap and abdominal straps and Velcro straps for the arm pocket can be trimmed for this purpose.

The patient should sit at a table when putting on and taking off the support, this prevents unwanted movements of the shoulder.

Recommend the patient to wear a T-shirt or undershirt under the product that sits without creases.

### Preparing the OmoLoc 15 shoulder orthosis:

(see back fold-out page)

- 1 First loosen the arm pocket (B), which is already fastened to the shoulder cushion (A) for treatment of the right arm, in case the left arm is being treated. Then position the cushion on the patient's left side and fasten the arm pocket to the flat surface. In this process, make sure that the large Velcro area on the cushion is facing outwards and upwards. The Velcro area must be fully covered by the arm pocket to prevent skin irritation due to the Velcro.
- 2 Fasten the two guide loops (F) to the cushion (A) with the loop facing upwards, pointing slightly diagonally towards the unaffected shoulder on the front and back (for orientation of front side vs. back side, see Bauerfeind word mark and figurative mark).
- 3 First pull the shoulder strap (E) with the square strap end from the outside through the guide loop (F) fastened to the rear on the cushion and refasten the strap. The rounded end of the strap thus lies on the side of the patient's chest. Thread the strap cushion (D) onto the strap.
- 4 Now take the abdominal strap (E) and also thread this with the square strap end from the outside through the guide loop already fastened to the rear of the cushion (A) and refasten the strap.

### Fitting to patients:

- 5 Position the cushion with the arm pocket on the affected side next to the patient. The cushion is positioned with the front area in an imaginary line to the front side of the body.
- 6 Place the patient's elbow into the opened arm pocket and position the elbow as far back as possible. The hand should be resting in the forearm pocket. In case the textile area is too long for the hand, the front area of the pocket can be folded over a bit.
- 7 Close the arm pocket by first attaching and fastening the Velcro strap to the cushion (A) at the inner elbow. The elbow rests in the arm pocket bent at a 90° angle. Then attach and fasten the second Velcro strap at the cushion (A) close to the wrist. Trim the square ends of the straps if they are too long or hang down.
- 8 Then guide the shoulder strap from the back to the front over the opposite shoulder, thread through the guide loop from the inside to the outside and fasten the strap.
- 9 By adjusting the length of the shoulder strap, position the cushion with the arm pocket roughly at waist height. When adjusting, ensure that both shoulders are at the same height afterwards. As needed for the individual patient, trim the shoulder strap at the square end in the back. Make sure that the double-sided Velcro is not cut up.
- 10 The strap cushion (D) is positioned correctly if the healthy shoulder is well cushioned.
- 11 Now guide the abdominal strap behind the back to the front, then thread the strap through the guide loop from the inside to the outside and fasten it. If necessary, the strap can be trimmed at the back on its square end. Here as well, make sure that the double-sided Velcro is not cut up.

### Instructions for patients

Before putting on the orthosis, the Velcro fastening of the abdominal strap should be loosened in the front as well as the two straps of the forearm pocket at the top of the cushion. Sit down when putting on the product and use the edge of a table to support your forearm. Ask someone to help you put on and take off the orthosis if necessary.

- 1 Slide your head through the shoulder strap so it is then positioned on the healthy shoulder.
- 2 Position the shoulder cushion so that its front side is positioned in an imaginary line to the front of your body. Place your elbow into the opened arm pocket and position it as far back as possible. The arm should rest with a 90° angle in the elbow.
- 3 Close the forearm pocket by first fastening the strap in the inner elbow on the cushion so that the elbow is properly positioned in the arm pocket. Then fasten the second strap on the cushion close to the wrist.
- 4 After putting on the product, check in front of a mirror to make sure that both shoulders are at the same height. Correct the height of the orthosis if necessary by adjusting the length of the shoulder strap using the Velcro fastening. Also check that the arm and shoulder have not rotated internally or externally to the torso.
- 5 Now guide the abdominal strap behind the back to the front, then thread the strap through the guide loop from the inside to the outside, pull it tight and fasten it.

## Cleaning instructions

**Note:** Never expose the product to direct heat (e.g. heaters, sunlight, in your car, etc.). This may cause damage to the material, which can impair the effectiveness of the product.

Please refer to the care instructions on the sewn-in label. This can be found on the upper edge of the arm pocket, in the elbow area:

1. Please use mild detergent and a lingerie bag for the cushion cover, the arm pocket and the straps. Wash the product regularly.
2. Close the Velcro fastenings of the product.
3. The product is not suitable for tumble drying.

## Part of the body this product is used for

In accordance with the indications (shoulder). Please see intended purpose.

## Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

## Assembly and fitting instructions

The product is supplied ready for use for treatment of the right arm. For treatment of the left arm, the product must be prepared by a specialist accordingly for your treatment.

## Technical specifications / parameters

The OmoLoc 15 is an orthosis for the shoulder joint. It consists of a shoulder cushion, a forearm pocket, an abdominal strap, a shoulder strap with strap cushion, guide loops and two Velcro straps for the arm pocket.

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the OmoLoc 15 have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical product to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Polyurethane (PUR), Polyamide (PA), Polyoxymethylene (POM), Elastane (EL), Polyester (PES), Polypropylene (PP), Synthetic resin, Styrene-ethylene / Butylene-styrene block copolymer (TPS-SEBS)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2021-03

<sup>1</sup>Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or torso

<sup>2</sup>A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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