 **BAUERFEIND AG**
Triebseser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-40 00
F +49 (0) 36628 66-44 99
E info@bauerfeind.com
BAUERFEIND.COM

EpiPoint®

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended use

EpiPoint is a medical device. It is an orthosis for the targeted relief of the tendon insertions at the elbow.

Indications

- Epicondylitis lateralis humeri (inflammation of the muscle origin, tennis elbow)
- Epicondylitis medialis humeri (inflammation of the muscle origin, golfer's and thrower's elbow)
- (Relapse) prevention

Risks of using this product

- **EpiPoint is a prescribed product that should be worn under a physician's guidance. EpiPoint should only be applied in accordance with these instructions for use and at the listed areas of application.**
- Should you notice any unusual changes or increased symptoms, please contact your physician without delay.
- No product liability is accepted in the event of improper use.
- The EpiPoint should not be altered arbitrarily.
- In the event of a product defect or wear to individual parts, please contact your medical retailer.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.

- The product's unique effect is produced primarily during physical activity. The product should be removed during prolonged periods of rest (e.g. sleeping, continuous sitting).

LATEX Contains natural rubber latex (elastodienne), which can cause skin irritation in persons with an allergy. Take off the orthosis if you notice any allergic reaction.

Contraindications

Side effects involving the body as a whole, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves.

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the forearm and the hand.
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin located remotely from the fitted aid.
4. Bottleneck syndromes (irritation due to compression of nerves in the arm).

Application instructions

Putting on the EpiPoint

Undo the band and place the EpiPoint strap on the forearm, ensuring the five-point pad rests directly on the muscle belly, i.e. in lateral position (outside) **1b** in case

of tennis elbow or in medial position (inside) **1b** in case of golfer's elbow. The rigid part of the strap should always face outwards. Position the pad approx. 4–6 cm below the elbow.

To fasten the strap, rest the forearm on a firm surface (table, etc.) and tighten the Velcro fastening. **2a** **2b** Please avoid fastening the orthosis too tight! If the red warning section indicates excessive contact pressure, make sure to loosen the strap accordingly.

Product care and washing instructions

Please refer to the care instructions on the sewn-in label of your EpiPoint.

EpiPoint can be washed separately by hand at 30 °C using a mild detergent. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaints, please contact your retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications (elbow). See purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Information about re-use

The EpiPoint is intended for your treatment only.

Warranty

The legal provisions of the country in which the product was purchased

apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

You can dispose of the product in accordance with national legal requirements.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.