 **BAUERFEIND AG**  
Triebseser Straße 16  
07937 Zeulenroda-Triebes  
Germany  
P +49 (0) 36628 66-4000  
F +49 (0) 36628 66-4499  
E info@bauerfeind.com  
**BAUERFEIND.COM**

## RhizoLoc®

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en english

## Dear Customer,

thank you for choosing a  
Bauerfeind product.

We work to improve the medical  
effectiveness of our products  
every day – because your health  
is very important to us. Please  
read through these **instructions  
for use** carefully. If you have any  
questions, please contact your  
doctor or medical retailer.

### Intended purpose

RhizoLoc is a medical device.  
It's an orthosis used for the  
stabilization of the metacarpal

phalangeal and carpometacarpal  
joints of the thumb.

### Indications

- Post-operative and post-  
traumatic irritation
- Collateral ligament injuries,  
e.g. skier's thumb
- Osteoarthritis of the carpometacarpal joint of the thumb (mild)
- Sprains / contusions

### Risks of using this product

#### Caution\*

RhizoLoc must only be worn in  
accordance with the specifica-  
tions contained in these instruc-  
tions and used for the areas of  
application (places of use) listed.

To guarantee the best possible fit  
of the RhizoLoc, it is essential to  
check whether the product is  
adequately shaped and whether  
it needs to be anatomically  
reshaped by a specialist<sup>1</sup>. Only a  
trained specialist may carry out  
the (initial) fitting of the orthosis  
and provide instruction.

The sewn-in label of the  
RhizoLoc, which contains  
information on the product  
name, size, manufacturer,  
washing instructions and CE  
certification, is located on  
Rhizobic outside of the wrist  
strap.

Use in combination with other  
products, e.g. as part of  
compression treatment (arm  
stockings), must be discussed  
beforehand with your physician.

Should you notice any unusual  
changes (e.g. increased symp-  
toms), please contact your physi-  
cian without delay. No product  
liability is accepted in the event  
of improper use.

Avoid the use of ointments,  
lotions or any substances

including grease or acids. Do not  
make any alterations to the  
product. Failure to comply with  
this requirement may adversely  
affect product performance,  
thereby voiding any product  
liability.

Side effects involving the body  
as a whole have not been  
reported to date. Correct fitting  
is assumed.

Do not put the RhizoLoc on too  
tightly as this could lead to  
local pressure symptoms. In  
rare cases, constriction of blood  
vessels and nerves may occur. If  
this happens, loosen the straps  
of the RhizoLoc and, if necessary,  
check the size of your RhizoLoc.

### Contraindications

Hypersensitivity reactions  
harmful to health have not been  
reported to date. In the following  
conditions such aids should only  
be fitted and worn after  
consultation with your physician:

- skin disorders / injuries in  
the relevant part of the body,  
particularly if inflammation is  
present. Likewise, any raised  
scars with swelling, redness  
and excessive heat build-up
- impaired sensation and  
circulatory disorders of the  
arm / hand (e.g. diabetes  
mellitus)
- impaired lymph drainage,  
including soft tissue swellings  
of uncertain origin located away  
from the fitted orthosis

### Application instructions

#### Putting on the RhizoLoc

**A – wrist strap, B – thumb strap,  
C – thumb tab**

- ① Open the two Velcro straps on  
the RhizoLoc. Loosen the straps  
without pulling them out of the  
guide. ② Put the RhizoLoc on the

affected thumb. To do this, guide  
your hand through the bottom  
wrist strap and your thumb  
through the top thumb strap.

- ③ Close the wrist strap. Then  
close the thumb strap by closing  
the Velcro at the side next to the  
thumb tab.

- ④ Following consultation with  
your physician, you can use the  
thumb tab to adjust the level of  
stabilization of the thumb basal  
joint depending on the phase of  
your treatment.

#### Taking off the RhizoLoc

Open the thumb and wrist straps.  
Guide your hand downward out  
of the RhizoLoc.

### Cleaning instructions

**Note:** Never expose the RhizoLoc  
to direct heat (e.g. heaters,  
sunlight, in your car, etc.!) This  
may damage the material, which  
can impair the effectiveness of  
the RhizoLoc.

Fasten all Velcro straps before  
washing to avoid damage. Wash  
the RhizoLoc separately by hand,  
using a mild detergent.

If you have any complaints,  
please contact your medical  
retailer only. We would point out  
that only clean goods can be  
dealt with.

### Part of the body this product is used for

In accordance with the indica-  
tions (thumb). See intended use.

### Maintenance instructions

If handled and cared for cor-  
rectly, the product is practically  
maintenance-free.

### Assembly and fitting instructions

The RhizoLoc is supplied in an  
anatomically contoured state. If

necessary, the RhizoLoc can be contoured further. The orthosis is available in right- and left-handed designs.

## Technical specifications / parameters

The RhizoLoc is an orthosis for the thumb. There are no spare parts available.

## Notes on reuse

The RhizoLoc is intended for your treatment only. It has been adapted specifically for you. For this reason, do not pass the RhizoLoc on to other people.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the RhizoLoc have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2020-03

## Specialists<sup>1</sup>

### General notes

- Check the optimal and individual fit of the RhizoLoc after the first fitting. Make corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.
- The RhizoLoc is intended for the treatment of one patient only.

The aluminum stays are anatomically contoured, but they may still need to be individually adjusted to suit the patient's anatomy. To do this, apply a little force to contour the aluminum stays into the anatomically correct form.

The thumb tab to limit the thumb basal joint must be firmly strapped in and under tension when treatment begins and must contain no folds. It is the physician's responsibility to adjust the level of restriction of movement of the thumb.

\* Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

<sup>1</sup> A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.