**medi GmbH & Co. KG**  
 Medicusstraße 1  
 D-95448 Bayreuth  
 Germany  
 T +49 921 912-0  
 F +49 921 912-780  
 ortho@medi.de  
 www.medi.de

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@medi australia.com.au  
 www.medi australia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: +32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

4 051526 890664

medi Canada Inc / médi Canada Inc  
 597, Rue Duvernay, Verchères  
 QC Canada J0L 2R0  
 T: +1 450-583-3317 / +1 800 361 3153  
 F: +1 888-583-6827  
 service@medicanada.ca  
 www.medicana.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbæk Strand  
 Denmark  
 T +45-70 25 56 10  
 F +45-70 25 56 20  
 kundeservice@sw.dk  
 www.medit danmark.dk

medi Bayreuth Espana SL  
 C/Canigo 2-6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34-932 60 04 00  
 F +34-932 60 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33-1 48 61 76 10  
 F +33-1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1 371-0090  
 F +36 1 371-0091  
 info@medi.hu  
 www.medi.hu

medi Japan K.K.  
 5-1 Nihonbashi Kabutocho Chuoku  
 103-0026 Tokyo  
 Japan  
 T: +81 3 6778 2590  
 F: +81 3 5847 7901  
 info@medi-japan.jp  
 www.medi-japan.co.jp

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31-76 57 22 555  
 F +31-76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi

# medi Epibrace

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкція по використанню. Navodila za uporabo. دليل الاستخدام. 使用说明. Instrucțiuni de utilizare. שימוש.

E013521/02.2021

medi. I feel better.

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## Important notes

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
Great Britain  
T +44-1432 37 35 00  
F +44-1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Evhen Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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Polyester, Polyamid, Polyurethan, Polypropylen

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# medi Epibrace

## Intended purpose

medi Epibrace is an epicondylitis brace for stress relief on the muscle origins.

---

## Indications

All indications where it is necessary to relieve muscle origins at the ulnar and / or radial epicondyle, such as:  
Epicondylitis (e.g. tennis or golfer's elbow)

## Contraindications

Impingement syndrome (pressure irritation of the nerves supplying the arm)

## Risks / Side Effects

If the aids are too tight, this can cause pressure sores or constriction of blood vessels or nerves. Consult your doctor before using this device if you have the following conditions:

- A condition affecting the application area
- Paraesthesia, circulatory disorders
- Lymph drainage disorders

Tightly fitting aids can cause skin irritation

(pressure, sweat, material).

## Intended patient groups

Healthcare professionals should provide care to the adults and children, applying the available information on the measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

## Note on wearing

During longer periods of rest (e.g. when sleeping) you should take the medi Epibrace off.

## Applying the strap

- Open the strap on the brace and lie it down, fully unrolling it, in front of you. Position your forearm on the brace with your palm facing upwards so that the light grey positioning mark on the pad is positioned on the side that you feel pain. Then attach the pad to the positioning mark so that the tip of the pad is parallel to the elbow joint (Fig. 1). Example: If you have tennis elbow (Epicondylitis humeri radialis) in your right arm, the positioning

mark should now be on the right side of your right arm.

- Thread the strap through the plastic loop and attach the fastening. The distance between the edge of the fastening and the outer elbow should be approx. 3 cm when the elbow is bent (Fig. 2).
- Refer to the table for guidance values on how tight the fastening can be done up (Fig. 3). Your forearm circumference (measured 5 cm below the elbow joint) should be used as the reference value. Tighten the strap until you attain the value shown on the chart. You can adjust the pressure according to your wishes.
- The clasp should now be firmly positioned on your arm, without slipping or constricting the arm. Form a fist with your hand. You should be able to feel your muscle under the pad. When your muscles are relaxed, the brace should sit against the pad without a noticeable feeling of pressure.
- In principle, the product should be used in accordance with medical supervision and should be fitted by a trained specialist, i.e. an orthopaedic technician, the first time that you put it on. If you wish to use other products

with this product, then this should be discussed and agreed with your GP.

- If your arm or hand becomes numb, loosen the aid or remove it if necessary. If your condition persists, consult a doctor.

---

### Care instructions

Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

---

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

### Material composition

Polyester, polyamide, polyurethane, polypropylene

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

### Your medi team

wishes you a quick return to full fitness.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the

Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code [UDI](#).

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