medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Cristóvão de Vita, n.º 260, Un. 30,
Centro Logístico Raposo Tavares,
Vila Camargo
Vargem Grande Paulista - SP
CEP 06730-000
Brasil
T: +55-11-3500 8005
sac@medibrasil.com
www.medibrasil.com

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

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Important notes
This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Polyamid, Polyester, Elasthan, Merino

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Levamed®

Intended purpose

Levamed is an ankle support with silicone pads. The product is exclusively to be used for the orthotic fitting of the ankle joint and only on unbroken skin or if any wounds have been properly covered.

Areas of Use

- Feeling of instability in the ankle joint (acute or chronic, after injuries to the ligament)
- Irritations in the joint (e.g. excessive stress, injuries, operations) in connection with swelling and bruising
- Inflammations in the joint (arthritis)
- Painful wear of the joint (arthrosis)

Side-Effects/Contraindications

There are no known side effects if the correctly sized product is fitted as recommended. If unexpected pain, swelling or numbness occur whilst wearing the support remove it and seek medical advice. Particular care is required where the following conditions are present:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as unclear swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression stockings) must be discussed in advance with the responsible doctor.

Mode of Action/Comment on Wear

Levamed causes a massage effect, which supports perfusion, with interaction between the compressive knitted fabric and the two integrated silicon inlays. This effect reduces swelling and bruising. Irritation and pain are relieved. Muscular control and the stabilisation of the ankle joint improve.

Supports are particularly beneficial during physical exercise. The supports are designed to fit comfortably and may be worn all day. It should be removed during prolonged periods of inactivity (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and seek medical advice.

The support is suitable for use in water. After use, please rinse thoroughly.

Putting on/Taking off

- Fold the top of the support over as far down as the silicone inlay (figure 1).
- Take hold off the support in the middle of the silicone inlays and pull it over the foot until the heel part fits tightly (figure 2).
- Unfold upper part of the support and pull it upwards, if necessary (figure 3). Ensure that the silicone inlays are correctly located either side of the ankle (figure 4).

Initial application must be made under medical supervision.

Care instructions

We recommend using a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort-effect and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, polyester, elastane, merino

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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