medi GmbH & Co. KG
 Medicusstraße 1
 D-95448 Bayreuth
 Germany
 T +49 921 912-0
 F +49 921 912-780
 ortho@medi.de
 www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o.o.
 Zygmunta Starego 26
 44-100 Gliwice
 Poland
 T +48-32 230 60 21
 F +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
 Room 105, Building No. 4,
 Lane 500 ShengXia Road
 Pudong New District
 201210 Shanghai
 People's Republic of China
 T: +86-21 61761988
 F: +86-21 61769607
 infocn@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Kültür mah. çaldıran sok.14/3
 06420 Çankaya/Ankara
 Turkey
 T +90 312 435 20 26
 F +90 312 434 22 82
 info@medi-turk.com
 www.medi-turk.com

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Polyamid, Polyester, Polyurethan,
Baumwolle

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Lumbamed[®] maternity

Intended purpose

Lumbamed maternity is a support brace. It is exclusively to be used for the orthotic fitting of the back during pregnancy and only on unbroken skin.

Indications

- Pain in the lumbar region during pregnancy
- Relief of complaints due to abdominal expansion
- Hyperlordosis due to pregnancy
- Lower back pain
- Lumbago combined with sciatica

Secondary effects / contraindications

Lumbamed maternity should not be used if the patient suffers from an inguinal hernia.

Other secondary effects which may occur in the entire body are currently unknown. However, if the supports are too tight, i.e. the bands, it can cause localized pressure sores or constriction of blood vessels or nerves in rare cases. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you: Disease or injury to the skin in the area that the brace is used on, particularly when signs of infection (excessive heat, swelling or redness) are apparent. Combining the brace with other products may be indicated, however, this must be approved by the doctor treating you.

Mode of action / how to wear the brace

Lumbamed maternity supports and alleviates stress on the tissue structures in the stomach, pelvis and the lumbar

region during pregnancy. The material wraps around the body and has integrated support slats in the lumbar region and tension can be adapted to the individual. All of this alleviates the increased stress on the region caused by the growing weight of the stomach, and pain is thus reduced.

Supports cause the greatest effect during physical exercise. In principal, the support can be worn all day. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and seek medical advice.

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Putting on the brace

Please wash the brace before its first use.

Please note that initial fitting should be supervised by a healthcare professional.

1. Before you put the brace on, loosen the fastening straps so that the brace can be put on easily. Release the belts' fasteners and fix them loosely at the back of the belt.
2. First, slip your hand into the hand strap on the right outer fastening side.
3. Wrap the brace around your body and place the left fastening side on your stomach, pulling it so that it is snug (Fig. 2).
4. Draw the right fastening side over the left one, and fasten it to the left side (Fig. 3).
5. Then release the fastening straps' fasteners and pull these to the front simultaneously (Fig. 4). You can adjust the tightness of the brace according to your needs and level of pain.

6. The brace is fitted correctly if the fastening is closed under the stomach and supports the stomach (Fig. 5).
-

Care instructions

Hook and loop fasteners should be closed for washing. We recommend using a laundry bag. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
 - Do not bleach.
 - Leave to dry naturally.
 - Do not iron.
 - Do not dry clean.
-

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, polyester, polyurethane, cotton

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi Team

wishes you a healthy and happy pregnancy!

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