medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
T: + 32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita,
 n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
T: +55-11-3500 8005
sac@medibrasil.com
www.medibrasil.com

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
T +45-70 25 56 10
F +45-70 25 56 20
kundeservice@sw.dk
www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
T +34-932 60 04 00
F +34-932 60 23 14
medi@mediespana.com
www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
T +33-1 48 61 76 10
F +33-1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
T +31-76 57 22 555
F +31-76 57 22 565
info@medi.nl
www.medi.nl

medi Orient M.E.A. ZCC
 TAURUS GROUP
 Suite No. M01 Wasel Al Mamzar Bldg.
 Al Wuhida Road, Al Mamzar
 P.O.BOX NO. 91464
 DUBAI, UAE
 United Arab Emirates
T: +971-4 2557344
F: +971-4 2557399
md@taurusgroup.net

medi Polska Sp. z o.o.
 Zygmunta Starego 26
 44-100 Gliwice
 Poland
T +48-32 230 60 21
F +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
T +351-21 843 71 60
F +351-21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

M.4s® comfort

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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M.4s[®] comfort

Intended Purpose

The M.4s comfort is a functional knee orthosis. The product is exclusively to be used for the orthotic fitting of the knee and only on intact skin or any wounds have been properly covered.

Indications

- Ruptures of the anterior or both cruciate ligaments with accompanying injuries
- Isolated ruptures of the anterior cruciate ligament
- Ruptures and/or instability of the collateral ligaments
- After re-construction of the anterior ligament
- For conservative treatment of knee joint instability
- Postoperatively after meniscus operation
- Postoperatively after cartilage operation

Contraindications

None known at present.

Setting the flexion / extension limits

Extension limit wedge, gold:

5° Hypext., 0°, 10°, 20°, 30°, 45°

Flexion limit wedge, silver:

0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°, 120°

Can be immobilised at:

0°, 10°, 20°, 30°, 45°

For immobilising the orthosis first insert the extension wedge and then the corresponding flexion wedge. The medial (inside) and the lateral (outside) wedge settings must always be the same.

1. 0° extension and 90° flexion wedges are pre-installed in your knee brace as standard (In the UK only 10° extension wedges fitted, remainder included as spare parts).
2. In order to change the wedges, press the joint cover on the rear side (see profile) outwards.
3. You can change the wedges by removing the tweezers from the joint cover and pulling the wedges forwards (extension) or backwards (flexion) out of the joint using the tweezers.

The wedges are accordingly labelled with „FLEX“ or „EXT“ and the corresponding degree. The wedges may also be distinguished by the colour. Extension wedges are golden, flexion wedges are silver. Push the new wedge from the front side (extension) or from the back (flexion) into the joint, so the markings „UP“ on the joint and the wedge are matched accordingly. Upon proper locking you will hear a click sound.

4. Move the joints of the orthosis to check the position of the wedges. Put the tweezers back into the mounting and attach the joint covers by hooking the two noses of the joint cover into the front side of the joint and then pushing the cover onto the joint. The cover is firmly attached if you hear a click sound.

Important notes

Only change the extension and flexion limits on instruction from your doctor or orthotist. In order to prevent hyperextension, always insert extension wedges or 0° wedges. The wedges define the exact excursion of the orthosis. In difficult soft-tissue conditions, e.g. where the soft-tissues are under compression, it may be necessary to insert the larger wedge to limit the joint movement to the desired excursion. In addition, make sure that the orthosis joints are correctly positioned: the middles of the joints should be level with the upper border of the patella. The joint must lie behind the midline of the leg.

Shaping the orthosis

When shaping the orthosis, please ensure that the joints stay parallel. This guarantees that the orthosis will function well and run easily and that there is no excessive wear.

Fastening the belt eyes

The belt eyes have been adequately fastened and are, to some extent, deliberately mobile. Please do not tighten the screws; they are tight enough. If you apply too much force, you can break off the screws.

Fitting instructions

- Open all the straps. Sit on the edge of a chair, pull the brace up the affected leg and flex your knee to 45°.
- Position the joints so that the centre of the joint (extension wedges' height) is positioned at the level of the top edge of the kneecap and slightly behind the leg's side central line. Please make sure that the brace has been placed on the leg straight and is not twisted. Fasten Belt No. 1 first and then Belt No. 2.
- Keep your leg bent and check again that you have put the brace on properly.
- First fasten the lower front calf strap (strap #1), then the upper back thigh strap (#2). Keep your knee in a flexed position and push both joints towards the back of the leg.
- Now fasten the lower back thigh strap (#3).
- Now fasten the upper front calf strap (#4). Then the upper back calf strap (#5), and finally the front thigh strap (#6).
- Now move the knee lightly a few times and check again that you have put the brace on properly. Make any necessary corrections.

Care instructions

The brace cannot be worn in saltwater. If the brace is worn in chlorinated water, it should be rinsed out thoroughly with clean water afterwards. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

With normal use, the hinge mechanism should not require any maintenance, but it can be lubricated with Teflon spray. Check regularly that the joint screws are secure, tighten them if necessary.

Storage instructions

Keep the brace in a cool, dry place and do not expose to direct sunlight.

What can I do if the brace slips?

- Check that the brace is fitted properly. The centre of the brace joints should be level with the upper border of the kneecap.
- Make sure that the back thigh strap (#5) is above the belly of the calf muscle
- Take the covers off the knee pads and wear the pads directly on your skin. Watch for skin reactions; skin irritation may occur in people with very sensitive skin.
- Do not wear the brace over clothes.

Material composition

Avional, Velours, Polyester, PU foam

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi Team

Wishes you a quick return to full fitness.

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