medi GmbH & Co. KG
 Medicusstraße 1
 D-95448 Bayreuth
 Germany
 T +49 921 912-0
 F +49 921 912-780
 ortho@medi.de
 www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

E+motion®

Manumed® active E+motion®

Handgelenkbandage
 Wrist support

Gebrauchsanweisung. Instructions for use. Mode d'emploi.
 Instrucciones de uso. Instrucoes para aplicacao. Istruzioni
 per l'uso. Navod k pouziti. Instrukcija po ispolzovanju.
 Instrukcja zakladania. Instrukcja z wykorzystaniem. Navod
 na pouzitie. دليل الاستخدام. 使用说明. Istruzioni di utilizzare.
 הוראות שימוש.

medi Performance Collection

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds and use it only as instructed by your doctor or orthotist.

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Manumed active

Intended purpose

Manumed active is a wrist compression support. The product is exclusively to be used for the orthotic fitting of the wrist and only on intact skin or if any wounds have been properly covered.

Areas of use

- Painful attrition of the wrist (arthrosis)
- Irritations and inflammations in the wrist (e.g. arthritis, rheumatic, through overstrain, after operations, etc.)
- Wrist injuries, e.g. sprains
- Tendon inflammations
- Feelings of instability in the wrist

Side-effects / Contraindications

There are no known side effects if the correctly sized product is fitted as recommended. However, if supports (or similar aids) are too tight, this may lead to local pressure effects or the constriction of blood vessels or nerves in individual cases. You should thus consult your doctor in the following circumstances and before applying the support,:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Paralgesia and circulatory disorders of the forearm or hand.
- Lymph outflow disorders as well as unclear swellings of soft tissues outside the area of use.

Any combination with other products (e.g. arm stockings) must be discussed beforehand with the attending doctor.

Mode of action / Comment on wear

Manumed active has a stabilising effect that prevents unhealthy movements of the wrist through the interaction of compressive knitted fabric, belt strap and individually adjustable splints. This relieves the muscles of the lower arm as well as tendons and tendon insertions in the affected area and reduces irritations. The compressive effect of the knitted fabric also restricts swelling.

In principle, the support can be worn for the whole day but should be taken off during longer periods of rest (e.g. sleep). But you should make this dependent on how you feel. In the event of unexpected, excessive changes, e.g. an intensification of the symptoms, please take the product off immediately and contact your doctor or supplying dealer.

The support is suitable for use in water. After use, please rinse out thoroughly.

Putting on / Taking off

Please note that initial fitting should be supervised by a qualified healthcare professional.

- Before you put the bandage on, stick the attached strap to the middle of the cloth-covered support. The wide end of the strap should be attached to the cloth-covered support so that the thumb opening in the bandage and the strap are on the same side (Fig. 1).
- Now pull the bandage over your hand. The “E+Motion” slogan should be on the back of the hand here (Fig. 2).
- Fasten the strap so that the opening in the strap goes over your outside wristbone (Fig. 3).
- Then attach it by affixing the narrow end of the strap to the strap (Fig. 4).
- Ensure that the support is positioned correctly when you put the bandage on. The support should be lined up with the underside of your forearm as straight as possible.

To take it off, undo the strap. Push the palm of your hand out of the bandage. Apply pressure to the inside pockets and guides, and pull the bandage off your hand.

Adjusting the splint

The splint can be individually adjusted at a temperature of around 200 °C.

Care instructions

Velcro fasteners should be closed for washing. We recommend using a laundry bag. The splint must be removed before washing. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort effect and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Note on storage

Must be stored dry and at room temperature. Must be protected from the sun and moisture.

Composition of the material

Polyamide, polyester, merino, elasthane

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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