medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-57
info@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: +32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita,
 n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

4 049772 348411

E011579/04.2018

protect.St pro

Kurze Soft Orthese
 Short knee orthosis
 Orthèse souple court
 Ortésis blanda corta

Gebrauchsanweisung. Instructions for use. Mode d'emploi.
 Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso.
 Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning.
 Návod k použití. Uputa za uporabu. Инструкція по використанню.
 Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής.
 Használati útmutató. Uputstvo za upotrebu. Návod na použitie.
 Instrucțiuni de utilizare.

medi

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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protect.St pro

Intended purpose

protect.St pro is a short knee soft brace. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin.

Indications

- Light to medium instabilities
- Chronic instability
- Meniscus injuries

Contraindications

None known at present.

Fitting instructions

- Open the straps and pull them back toward the back so that they will not disturb you putting on the brace.
- Now put your leg into a 45-degree angle flexion position. Pull the brace up over you leg (Image 1).
- Make sure that the kneecap is positioned centrally in the space for the patella. Position the brace so that the middles of the hinges are level with the upper border of the kneecap (Image 2).
- Now fasten both straps and first tighten the calf strap and then the thigh strap (Image 3).
- If possible, please walk a few steps in order to check whether the brace is in the correct position (Image 4). Please adjust the position as required.
- In order to take off the brace, take hold of it at the lateral, lower end and pull it downward (Image 5).

Care instructions

Please close the Velcro fasteners before washing and remove the orthoses. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.

- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the brace in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, Elastan, aluminum

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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