Obsah

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PLEASE READ CAREFULLY BEFORE
USING WHOLE OF INSTRUCTIONS

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Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasize that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

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Walking aid designed for disabled people for those with locomotor disorders.

Ideal for rehabilitation process and physiotherapy (verticalisation, lower limb relief).

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

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- The product is packed in folded position.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above (Illustration 1).
- Ensure that all components have no visible damage. If any doubts, please contact the distributor.

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- In order to unfold the walker, should leaning its sides maximally outside till the moment of latching the blockade. (Illustration 3.)
- For easy storage and transport, the walker can be folded. To do that, put down the plastic submission button on the reinforcing frame.
- Fold both sides of the frame thereafter. Release submission button. Walker should be folded (Illustration 2).
- Walker can be moved, holding by side frames.

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the execution of a walk movement and submission of walker frames.

- For easy storage and transport, the walker can be folded.
- To do that, should put lock pin moving function in the reinforcing frame.
- Than push the submission button placed on the frame and fold side frames.
- Release submission button. Walker should be folded to position shown on the Illustration 2.
- Walker can be moved, holding by side frames.

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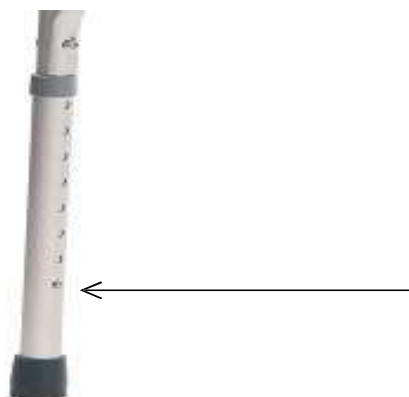
HEIGHT ADJUSTMENT

- Before using a walker, should adjusted to the user's height by telescopically adjustable feet length.
- To do this, press the height adjustment pin of the legs and depending on the needs reducing the height of the legs or extend the leg (Illustration 6).
- It is important to the length of all legs was the same (handles must be parallel to the ground).
- The handle should be at the height that will provide comfort and a sense of security for the user.

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MOVING FUNCTION

- The moving function is enabled when the pins lock are located in the reinforcing frame (Illustration 4).
- Please stand nearby walker, holding the handle with both hands.
- In order to use the device, interchangeably lift up the walker's lateral parts and move them forward and then put feet.

CONSTANT FUNCTION

- In order to get walker constant function should remove the lock pin moving function from reinforcing frame and put it in lock position walk function and walker folding (Illustration 5).
- In order to use the walker constant function, lift it up, move forward and approach it.
- In order to get the moving function or fold the walker, put lock pin in the reinforcing frame.

CARE AND MAINTENANCE

- Clean the walker frame with a damp cloth.
- Remove contaminants on a regular basis
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly.
- Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic.
- Do not expose the walker on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

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TECHNICAL REVIEW

- The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Every two weeks	Screw, nut and rivets control regarding their correct fixing.	Make sure if all the screws, top and rivets are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.

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Repair

- In case of repair, please contact the distributor.

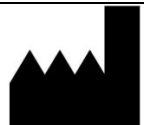
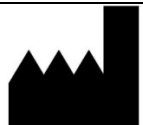
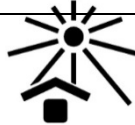
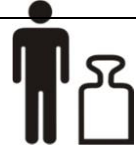
Customer service

- If you have any questions or need help, contact the distributor.

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DISTRIBUCE A SERVIS / DISTRIBUTION AND SERVIS

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