ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

Carpal tunnel syndrome, Tendinopathies/Teno-synovitis

PERFORMANCE CHARACTERISTICS

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS

• None

WARNINGS AND PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

- 1) Open all the straps and slide hand into brace so that the straps are on the outside of the hand
- 2) Attach middle strap around wrist and then fasten the bottom strap around the arm.
- 3) Finally thread the upper strap through the brace and then wrap between thumb and finger and fix to brace. Adjust the strap to fit comfortably.
- 4) Manipulate brace to fit hand comfortably, but ensure the carpal tunnel is exposed through the open section of brace.

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION

Polyethersulfane, Elastane, Thermal Polyurethan, Polyamid, Polyurethan, Aluminium, Polyester

NOT MADE WITH NATURAL RUBBER LATEX.

FOR SINGLE PATIENT USE ONLY

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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