medi GmbH & Co. KG  
Medicusstraße 1  
D-95448 Bayreuth  
Germany  
F +49 921 912-0  
F +49 921 912-57  
info@medi.de  
www.medi.de

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
P +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium NV  
Staatsbaan 77/0099  
3945 Ham  
Belgium  
T: +32 11 24 25 60  
F: +32 11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Canada Inc / médi Canada Inc  
597, Rue Duvernay, Verchères  
QC Canada J0L 2R0  
T: +1 450-583-3317 / +1 800 361 3153  
F: +1 888-583-6827  
service@medicanada.ca  
www.medicanada.ca

MAXIS a.s.,  
medi group company  
Slezská 2127/13  
120 00 Prague 2  
Czech Republic  
T: +420 571 633 510  
F: +420 571 616 271  
info@maxis-medica.com  
www.maxis-medica.com

medi Danmark  
Vejlegardsvej 59  
2665 Vallensbaek Strand  
Danmark  
P +45 70 25 56 10  
F +45 70 25 56 20  
kundeservice@sw.dk  
www.meditandmark.dk

medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospitalet de Llobregat  
08901 Barcelona  
Spain  
P +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
P +33 1 48 61 76 10  
F +33 1 49 63 33 05  
info@medi-france.com  
www.medi-france.com

medi Hungary Kft  
Bokor u. 21.  
1037 Budapest  
Hungary  
P +36 1 371 0090  
F +36 1 371 0091  
info@medi.hu  
www.medi.hu

medi Italia S.r.l.  
Via Giuseppe Ghedini, 2  
40069 Zola Predosa (BO)  
Italy  
P +39 0 51 613 24 84  
F +39 0 51 613 29 56  
info@medi-italia.it  
www.medi-italia.it

Tachezy Sanit Srl.  
Via Druso 48/a  
39100 Bolzano  
Italy  
T: +39 0471 288 068  
F: +39 0471 282 773  
info@tachezysanit.com  
www.tachezysanit.com

medi Japan K.K.  
5-1 Nihonbashi Kabutocho Chuoku  
103-0026 Tokyo  
Japan  
T: +81 3 6778 2590  
F: +81 3 5847 7901  
info@medi-japan.jp  
www.medi-japan.co.jp

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
P +31 76 57 22 555  
F +31 76 57 22 565  
info@medi.nl  
www.medi.nl

medi Norway AS  
Vestheimvegen 35  
4250 Kopervik  
Norway  
T +47 52 84 45 00  
F +47 52 84 45 09  
post@medinorway.no  
www.medinorway.no

medi

mediven®

Gebrauchsanweisung. Instructions for use.  
Mode d'emploi. Instrucciones de uso.  
Instruções de utilização. Istruzioni per  
l'uso. Gebruiksaanwijzing.  
Brugsanvisning. Instrukcija obsługi.  
Инструкция по использованию  
Návod k používání. upute za upotrebu.  
kullanım kılavuzu. Οδηγίες χρήσης της.  
használati útmutató. інструкція з  
використання. Navodila za uporabo za.  
návod na použitie. Instrucțiuni de utilizare.  
ръководство за употреба. שומיש תוארוה.  
kasutusjuhend. käyttöohje. lietošanas  
instrukcija. naudojimo instrukcija.  
使用说明. ماخذتسالا تامي لعت.

406173665555

E016030/08.2020/200\*

medi. I feel better.

mediven®

Gültig für folgende mediven-Qualitäten / Valid for the following models / Valable pour les modèles suivants / Válido para las siguientes calidades / Válido para os seguintes modelos / Valido per i seguenti modelli / Geldig voor de volgende soorten / Gælder for følgende kvaliteter / Gäller för följande kvaliteter / Platí pro následující kvality / Važi za sljedeće kvalitete / Действительно для следующих видов продукции / Aşağıdaki kaliteler için geçerlidir / Obowiązuje dla następujących rodzajów / Ισχύει για τα ακόλουθα είδη / Érvényes a következő változatokra / Дійсно для наступних якостей / Veljavno za naslednje vrste / Platí pre nasledujúce kvality / اعوان الال على ىرسى / اىلالتال: / 适用于以下品级 / 適用於以下品質 / Valabil pentru următoarele calităţi / מיאבה מימדל רקת / Gyldig for følgende kvaliteter / 以下のクオリティーに適用 / Важи за следните качества / Kehtib järgmistele mudelitele / Voimassa seuraaville malleille / Attiecas uz šādiem veidiem / Taikoma šioms rūšims:

- mediven elegance
- mediven for men
- mediven comfort
- mediven plus
- mediven active
- mediven forte
- mediven angio
- mediven 550 leg
- mediven mondi
- mediven sensoo
- mediven cosy

Inhalt / Content

Deutsch / German	4 – 7
Englisch / English	7 – 10
Français / French	10 – 14
Español / Spanish	14 – 17
Português / Portuguese	17 – 21
Italiano / Italian	21 – 24
Nederlands / Dutch	24 – 27
Dansk / Danish	27 – 30
Svenska / Swedish	30 – 33
Čeština / Czech	33 – 36
Hrvatski / Croatian	36 – 39
Русский / Russian	39 – 43
Türkçe / Turkish	43 – 46
Polski / Polish	46 – 50
ΕΛΛΗΝΙΚΑ / Greek	50 – 53
Magyar / Hungarian	53 – 56
Українська / Ukrainian	56 – 60
Slovenščina / Slovenian	60 – 63
Slovenčina / Slovakian	63 – 66
ET / Arabic	66 – 69
中文 / Chinese	69 – 72
Română / Romanian	73 – 76
תירבע / Hebrew	76 – 78
Norsk / Norwegian	78 – 81
Български / Bulgarian	82 – 85
Estonian / Eestlane	85 – 88
Finnish / Suomalainen	88 – 91
Latvian / Latvietis	91 – 94
Lithuanian / Lietuvis	94 – 97


A


B

C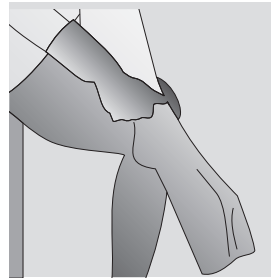

D

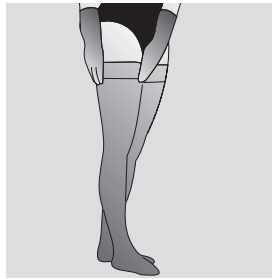E

**Symbolerläuterungen / Explanation of the symbols / Explication des symboles / Explicación de símbolos / Explicação de símbolos / Spiegazione dei simboli / Verklaring van de symbolen / Symbolforklaring / Symbolförklaring / Vysvětlení symbolů / Objasnjenje simbola / Пояснение символов / Açıklanan simgeler**  
**Objasnienie symboli / Επεξήγηση συμβόλων / Szimbólumok magyarázata**  
 Пояснення символів / **Razlaga simbolov / Vysvetlenie symbolov / زومرلل یتاعم**  
**符号说明 / 符號說明 / Explicare simboluri / יללמסה לע רבסה / Forklaring av symboler**  
 Обяснение на символите / **Sümbolite selgitus / Symbolien selitykset / Simbolu skaidrojums / Simbolių paaiškinimas**

**MD**

Medizinprodukt / Medical product / Produit médical / Producto médico / Dispositivo médico / Prodotto sanitario / Medisch product / Medicinsk produkt / Medicinteknisk produkt / Zdravotní výrobek / Medicinski proizvod / Медицинское изделие / Tibbi ürün / Produkt medycyny / Ιατροτεχνολογικό προϊόν / Orvostechnikai eszköz / Медицинский продукт / Medicinski pripomoček / Zdravotnícky produkt / ېب ط جتنم / 医疗器械 / 醫療產品 / Produs medical / יאופר רושכימ / Medisinsk produkt / Медицинско изделие / Meditsiiniseade / Lääkinnällinen tuote / Medicínska ieřice / Medicinos priemonė

**UDI**

Eindeutige Produktidentifizierung / Unique product identification / Identification unique du produit / Identificación clara del producto / Identificação única do produto / Identificazione unica del prodotto / Duidelijke productidentificatie / Entydig produktidentificering / Entydig produktidentificering / Jednoznačná identifikace výrobku / Jednoznačna identifikacija proizvoda / Однозначная идентификация изделия / Kesin ürün tanımlama / Jednoznacna identyfikacja produktu / Σαφής αναγνώριση προϊόντος / Egyértelmű termékazonosítás / Чітка ідентифікація продукту / Enoznačna identifikacija izdelka / Jednoznačná identifikácia produktu / جتنم لل حص او دي دجت / 医疗器械唯一标识 / 明確的產品標識 / Identificare univocă a produsului / ירוחיי רצומ ההזמ / Entydig produktidentifikasjon / Унікална ідентифікація на изделието / Unikaalne tootetähistus / Tuotteen yksiselitteinen tunnistus / Viennoztímiga produkta identifikacija / Aiškus priemonės identifikavimas

Mehrmalige Anwendung durch einen Patienten (s. auch 6. Trage- und Nutzungsdauer) / Multiple uses by one patient (see also 6. Length of time that compression clothing should be worn and used for) / Plusieurs utilisations par un patient (voir également 6. Durée de port et d'utilisation) / Uso repetido por parte de un paciente (véase también 6. Duración y tiempo de uso) / Reutilização por um único paciente (ver também a secção 6 Período de utilização e vida útil) / Utilizzo ripetuto da parte di un paziente (vedere anche Paragrafo 6 Durata dell'utilizzo) / Meermalig gebruik door een patiënt (zie ook 6. Draag- en gebruiksduur) / Gentagen brug ved samme patient (se også 6. Bære- og anvendelsestid) / Flergångsbruk av en patient (se även 6. Användningstid) / Vícenásobné použití jedním pacientem (viz také 6. Délka nošení a používání) / Višekratna upotreba jednog pacijenta (v. i 6 Nošenje i trajanje upotrebe) / Многократное использование одним пациентом (см. также раздел 6 «Продолжительность ношения и использования») / Bir hasta tarafından çok kez kullanım (ayrıca bakın 6. Giyme ve kullanım süresi) / Wielokrotne stosowanie przez jednego pacjenta (patrz także 6. Okres noszenia i użytkowania) / Επανελημμένη χρήση από έναν ασθενή (βλ. επίσης 6. Διάρκεια χρήσης) / Egy beteg általi többszöri használat (lásd még: 6. Viselési és használati időtartam) / Багаторазове використання одним пацієнтом (див. також 6. Час носіння та використання) / Večkratna uporaba na enem pacientu (gl. tudi 6. točko Čas

medi Polska Sp. z o.o.  
ul. Łabędzka 22  
44-121 Gliwice  
Poland  
T: +48-32 230 60 21  
F: +48-32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.  
Rua do Centro Cultural, no. 43  
1700-106 Lisboa  
Portugal  
T: +351 21 843 71 60  
F: +351 21 847 08 33  
medi.portugal@medibayreuth.pt  
www.medi.pt

medi RUS LLC  
Business Center NEO GEO  
Butlerova Street 17  
117342 Moscow  
Russia  
T: +7-495 374 04 56  
F: +7-495 374 04 56  
info@medirus.ru  
www.medirus.ru

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mediven®

### Instructions for Use

Dear Customer, Thank you for placing your trust in us! We're delighted that you've chosen medical compression stocking from medi. To ensure that your compression therapy is effective, please take a moment to read through these instructions for use carefully. Ensure that you store this important information carefully so that you can also refer back to it at a later point in time. Should you have any questions, please consult the doctor treating you, the medical professionals that you have sought care from or the medi Service team (see reverse).

#### 1. Intended use - what is medical compression stocking used for?

Round- or flat-knitted medical stockings are used for compression of the lower extremities, mainly for the treatment of disorders of the venous and lymphatic systems

#### 2. Features - how does medical compression stocking work?

Medical compression stocking works using a pressure gradient (decreasing from the end of your limb towards your heart) to exert compression on your limbs.

#### 3. Indications - which conditions is medical compression stocking used to treat?

You have received compression stocking from medi following your diagnosis. Please follow your doctor's orders and/or the orders of your medical specialist in order to obtain optimal therapeutic results. Please note that each item of compression stocking has differing indications and applications due to their differing material properties, how they were manufactured and the compression classes that they are available in.

#### In general, your mediven compression stocking can be used for the following indications:

##### Chronic venous diseases

- To improve venous symptoms
- To improve quality of life in chronic venous diseases
- Prevention and therapy of venous oedema
- Prevention and therapy of venous skin changes
- Eczema and pigmentation
- Lipodermatosclerosis and atrophic blanche
- Therapy of the venous ulcer
- Therapy for mixed (caused by arterial and venous issues) ulceration (taking the contraindications into account: see item 4)
- Prevention of recurrent venous ulcer
- Pain reduction for venous ulcer
- Varicose veins
- Initial phase following varicose vein therapy
- Functional venous insufficiency (with obesity or jobs that involve a lot of sitting or standing)
- Venous malformations

##### Thromboembolic venous diseases

- Superficial venous thrombosis
- Deep vein thrombosis in the leg
- Conditions that follow on from thrombosis

- Post-thrombotic syndrome
- Thrombotic prophylaxis in mobile patients

#### Oedema

- Lymphoedema
- Oedema during pregnancy
- Post-traumatic oedema
- Post-operative oedema
- Post-operative reperfusion oedema
- Cyclic idiopathic oedema
- Lipoedema
- Conditions involving stasis due to immobility (arthrogenic congestive syndrome, paresis and partial paresis of the extremity)
- Oedema caused by job (jobs which involve a lot of sitting or standing)
- Oedema caused by taking medication which cannot be altered (no viable alternative)

#### Other indications

- Obesity with functional venous insufficiency
- Inflammatory dermatitis of the leg
- Nausea, dizziness during pregnancy
- Complaints involving stasis during pregnancy
- Conditions resulting from burns
- Scar treatment

#### Please observe the following instructions:

Due to the different manufacturing methods used for medical compression stocking and the different modes of action that they elicit, medi generally recommends that round-knitted compression stocking (stocking without seams) be used for venous conditions and flat-knitted compression stocking (stocking with seams) be used for lymphatic system conditions. However, flat-knitted stocking may still be an appropriate therapy for a venous condition, for example (e.g. if the limb is very large at one point and narrow at another or if tissue folds are very deep). Note that this decision can only be made by a doctor.

Patient-dependent factors in particular, such as the patient's weight, the type and severity of the oedema and the nature of their connective tissue all play a role in this decision.

Therefore, our recommendation is as follows:

- The higher the patient's weight,
  - The more they tend towards oedema,
  - The more severe the disease,
  - The softer their connective tissue,
- > The more compressive the stocking material needs to be!

Your doctor or the medical professionals treating you will support and advise you in selecting the correct medical compression stocking for you. This ensures that your individual needs are met, which increases your wellbeing and provides you with the best chances of therapeutic success.

#### 4. Contraindications - are there instances in which I should not wear the medical compression stocking?

Medical compression stocking may not be worn if the patient has the following conditions:

- Advanced peripheral arterial occlusive disease (if any of these parameters is present: ABPI < 0.5, ankle arterial pressure < 60 mmHg, toe pressure < 30mmHg or TcPO<sub>2</sub> < 20 mmHg on dorsum of foot).

When non-elastic materials are implemented, the patient can still try out compression stocking if they have ankle arterial pressure between 50 and 60 mmHg and they are under close clinical supervision.

- Decompensated heart failure (NYHA III + IV)
- Septic thrombophlebitis
- Phlegmasia cerulea dolens

In the following instances, the therapeutic decision should be made by weighing up the benefits and risks and by selecting the most suitable compressive equipment:

- Severe weeping dermatosis
- Intolerance to compressive material
- Severe paraesthesia in the limbs
- Advanced peripheral neuropathy (e.g. in diabetes mellitus)
- Primary chronic polyarthritis

If you are not sure whether one or more of these statements applies to you, speak to your doctor and/or the medical personnel treating you.

The round-knit medical compression stockings, **mediven angio**, have product characteristics which are specially formulated to ensure that they are optimal for serving the needs of patients with concomitant conditions such as diabetes mellitus or mild to middle peripheral arterial occlusive disease (ABPI > 0.6). **mediven angio** has been confirmed to be safe for this patient population in a clinical study.<sup>1</sup>

The following risks and the following side effects may occur:

Medical compression stocking may incur the following, especially if handled incorrectly:

- Skin necrosis and
- Pressure damage to peripheral nerves.

If your skin is sensitive, the compressive equipment may cause itchiness, peeling or signs of inflammation. Adequate skincare under the compression stocking is vital. Therefore, please take particular care to observe our important notes and the instructions for putting the compression stocking on (sections 7 and 8).

If the following symptoms are experienced - the compression stocking should be removed immediately and clinical advice sought: Change in toe colour - blue or white, new experience of paraesthesia or numbness in toe/foot or leg, increase of pain, shortness of breath, sudden sweating or acute limitation in movement of limb.

<sup>1</sup> Rother U et al. Safety of medical compression stockings in patients with diabetes mellitus or peripheral arterial disease. *BMJ Open Diabetes Res Care* 2020;8(1):e001316.

#### 5. Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## 6. How long should my mediven compression stocking be worn for? How long can it be used for?

Unless otherwise specified by the doctor, wear your mediven compression stocking from morning to night for optimal therapeutic results. If the maximum wearing time is exceeded by a significant amount in exceptional cases (e.g. long haul flights), the top band should be moved frequently if you are wearing a thigh-length stocking, or you should consider wearing tights as an alternative.

A precise pressure gradient (high pressure at bottom, decreasing towards the top) is required to ensure that compression stocking functions optimally. Through daily wearing and washing, the medically necessary pressure and elasticity of your compression stocking can decrease over time. The stockings should therefore only be used for a maximum of six months. If the medical compression stocking has been used for six months and a repeat prescription is issued, the patient's dimensions must be checked again by the medical specialist retailer as they could have changed due to the patient's condition or their individual circumstances.

Observe our instructions for putting on the stockings and instructions for care to ensure that the compressive effect is maintained as well as possible (sections 8 and 10). Please do not hesitate to get in touch with us. We can give you information on the options for ensuring long-term effective therapy for venous and lymphatic system disorders using medical compression garments.

### 7. Instructions for use - what should I know?

- The compression stocking should be issued by trained specialists with a briefing on how to handle it and how to care for the compression stocking (see sections 8 and 10 on this).
- If pain or increased irritation of the skin occurs while you are wearing the compression stocking, remove it immediately and consult your doctor or the medical specialists.
- Appropriate skincare must be maintained during compression therapy. This is essential. Creams or salves that are particularly greasy and also soap residue can cause skin irritation and material wear and can thus impair the efficacy of the compression stocking. Therefore, medi also offers a skincare range (medi day, medi night, medi soft foam lotion) that is specially formulated for compression stocking. Please visit a medical specialist retailer to discuss which product is right for you.
- Compression stockings with a silicone top band can cause skin irritation in people with sensitive skin (caused by perspiration in combination with mechanical stress). In order to prevent this, it is recommended that the top band is repositioned slightly several times throughout the day. If the patient uses unsuitable skincare (see also previous point) or has a lot of body hair, then the compression stocking could slip. Please visit a medical specialist retailer to discuss this, should it occur.
- Sharp fingernails and jewellery can cause damage to the knit fabric.
- Do not attempt to repair your compression stocking yourself or to have it repaired by an external service provider, as this will render the guarantee void, with-

out exception. Furthermore, this can impair the quality, safety and efficacy of the product. You can obtain information on correct repair by medi - if possible - from your medical specialist retailer or our services.

- In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).
- The sewn-in fabric label is an important component in ensuring that the identification and traceability of the product is guaranteed. Therefore, ensure that it is not removed from the compression stocking under any circumstances. Do not tear it out or cut it out, as this renders any claim to the guarantee, repair or exchange null and void.

### 8. Care instructions - what do I need to do when putting on my mediven compression stocking?

Explanatory videos with tips and tricks for putting on the stocking correctly can be found here:

Ideally, you should put the compression stocking on immediately after you wake up. If you have a shower in the morning and then apply skin care products (see "Section 10 - Skin care products" here), then you should wait a while until your skin is completely dry. This will make it far easier to put on the compression stocking. Please also ensure that you have your medi fabric gloves or rubber gloves ready. These are available from specialist medical retailers. You will use them later. They make it easier to distribute the fabric over the leg, or you can use them to put the stocking on, right from the beginning. The gloves make it easier to put on the stocking.

*Tip: Ask about useful donning aids such as the medi Butler, medi 2in1 and medi slide.*

When you are putting the stocking on, ensure that your fingernails have been filed into a smooth, rounded shape. It is also important to ensure that your heels and toenails are well cared for to avoid causing damage to the knitted fabric. Only put your jewellery or watch on once you have put the compression stocking on. To ensure that your compression stocking is easy to put on, sit down and place your foot on an elevated support, such as a small stool.

*Tip for compression stockings with open toe: First, place your foot in the attached donning aid.*

**A.** Reach into the compression stocking and grasp the heel. Hold the heel and turn the compression stocking inside out.

**B.** The foot part (which has remained inside the compression stocking) now forms an opening. Stretch the opening out slightly using both hands.

**C.** Slide your foot into the opening (if the stocking has an opening in the toe, do this using the donning aid that comes with the product) and pull the compression stocking over your foot carefully, up to the heel.

**D.** Then distribute the material evenly over the leg without causing wrinkles. Do this by pulling it up the leg

bit by bit, ensuring that the compression stocking is not overstretched. In the interim, check that it is correctly fitted on the toe and heel. If required, correct this once you have put it on by pulling the compression stocking down slightly (tip: this is easier if you use the gloves).

**Tip for compression stockings with open ends:** After this step, remove the donning aid by folding the foot part of the compression stocking up towards the heel and pulling the donning aid out. After this, pull the foot component over your foot again and spread the knitted fabric evenly up to the start of the toes.

E. Finally, check the position of your compression stocking. A knee stocking should be positioned around two centimetres below the back of the knee, a thigh stocking should be positioned around two centimetres under the fold of the buttock. Tights should be comfortable around the waist and the gusset should fit securely around the crotch.

**Tip for thigh stockings:**

To protect the seam, grip the knitted fabric carefully when putting the compression stocking on - do not grip the seam of the top band.

**Tip for compression tights:**

Turn the pair of tights completely inside out. Then complete steps B and C for each side. After this, distribute the material on both sides evenly and in stages, first pulling up one side a little, then the other, until the tights are pulled up to your waist.

**Tip for taking off the stocking:**

Fold the compression stocking down in stages until you reach the heel. Ensure that you switch between the sides when removing stockings (lower one side slightly first, then the other). When you reach the heel, grip the knitted fabric and pull it over the foot. If the stockings are ankle stockings, do not tug on the top band. Ask your medical specialist retailer about handy doffing aids such as the medi Butler Off or the medi 2in1. They make it far easier to take off the compression stocking.

**9. Storage instructions - how long can I store my mediven compression stocking, and how should I store it?**

The products must be stored in a dry place and out of direct sunlight and heat.

- The hourglass symbol shown on the box label or on the measurement chart shows the latest date that the compression clothing can be used up until. For standard products the expiry date has been calculated by adding the maximum storage duration for standard products (36 months) and the maximum amount of time that the stocking can be used for (6 months).
- Models that have been tailored for the patient and made-to-measure versions are intended for immediate use. Therefore, the expiry date is calculated by adding the production, transportation and delivery duration (one month) and the maximum amount of time that the stocking can be used for (six months).

**10. Care instructions - what do I need to take note of in order to maintain my mediven compression stocking optimally?**

- Wash your compression stocking every day after you have worn it. We recommend that you use medi

clean or a specially formulated hand wash detergent to ensure gentle care. Alternatively, you can use washing detergent for delicates to wash the clothing by hand or in the machine (delicates cycle at 40°C) without optical bleach or conditioner.

**Tip:** A net washing bag offers extra protection for the knitted fabric.

- We recommend that you turn your compression stocking inside out when washing it. Wash it separately or with clothing that is similar in colour.
- If the compression stocking is stained, we recommend that you use medi spot ex. Please do not dry clean mediven compression stocking or clean it with household agents.
- After washing, you can roll the compression stocking up in a towel and press it to dry it. Do not leave the damp compression stocking in the wet towel, instead let it dry in the air. Do not place it in sunlight or on a radiator/heating appliance.
- Alternatively, you could also dry your mediven compression stocking in your tumble dryer using the delicates cycle.
- Stretch compression stocking with a seam (flat-knitted stocking) back into shape after washing.

---

	Wash up to 40°C (delicates cycle)
	Do not bleach
	Tumble dry at low temperatures (delicates cycle)
	Do not iron
	Do not dry-clean

---

**11. Material composition - which ingredients does my mediven compression stocking contain?**

- All mediven compression stockings contains polyamide and elastane. You can find more detailed information on the fabric label sewn into your compression stocking.
- Latex free: This product does not contain any rubber that has been obtained from the latex tapped from tropical plants.
- mediven comfort and mediven for men contain lanolin. Lanolin is a natural wool wax used in skin care products.

**12. Disposal - how should I dispose of my mediven compression stocking?**

- Please dispose of your medical compression stockings via your general waste or household waste. There are no specific criteria for disposing of the stockings.

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