**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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Polyamid (PA), Polyurethan (PUR), Polyether,  
Polyoxymethylen (POM), Polypropylen (PP), Polyester (PES)

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

## Intended use

The OmoLoc is a medical product. It is an orthosis for immobilization of the shoulder.

## Indications

- Immobilization, e.g. after shoulder surgery
- Post-operative, post-traumatic, pre-operative care
- Dislocation

## Risks of using this product

**Please closely observe the specifications in these instructions for use and the instructions given by the specialist.**

- OmoLoc must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application (intended use, parts of the body) listed.
- To ensure the optimum fit of the OmoLoc, the size of OmoLoc required must be determined by a trained specialist.
- The sewn-in label of the OmoLoc, which contains information on the size, manufacturer, washing instructions and CE mark, is located on the inside of the abdominal strap. Only a trained specialist may carry out the (initial) fitting of the support and provide instruction.
- The use and duration of use must be discussed with the physician who prescribed the product. They will also inform you whether the OmoLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the product as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the product – e.g. an increase in discomfort – stop using it and contact your physician without delay.

## Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness and excessive heat build-up
- Impaired sensation and circulatory disorders of the legs (e.g. diabetes mellitus)
- Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion)
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid

## Application instructions

A – Abdominal strap	D – Hand sling
B – Upper arm strap	E – Finger pocket
C – Forearm strap	F – Narrow side straps

The following instructions describe how to put on an OmoLoc orthosis to treat the right shoulder.

## Putting it on

- 1 Place the fingers of your right hand in the finger pocket of the right-hand fastening. Bend your right arm at a right-angle in front of your body. While doing this, hold the orthosis firmly under your arm with your elbow.
- 2 Now guide the orthosis around your back from behind. Ask someone to help you with this if necessary. Place the fingers of your left hand in the finger pocket of the left-hand fastening.
- 3 Now attach the right-hand fastening to the left with moderate tension.
- 4 Turn the OmoLoc around the trunk until the upper arm strap is on the right side of the body.
- 5 Now tighten both narrow side straps. The orthosis should now fit tightly but not uncomfortably on the body at waist height.
- 6 Undo the upper arm and forearm straps and the hand sling.
- 7 First secure the upper arm to the right side with the upper arm strap. Place the forearm and hand in the two remaining straps in front of the abdomen. When fastening the straps, make sure that the arm is held securely but that the straps are not too tight. If necessary, adjust the height of the abdominal strap on the body, so that the arm rests in a relaxed, comfortable position.

### Changing the strap positions (with the help of another person)

If necessary, the strap positions can be adjusted to suit the length of the forearm. **8** Remove a strap from the inside of the orthosis. Change its position and attach the strap to the inside again. The upper edge of the Velcro should line up with the upper edge of the abdominal strap. **9** The buckle is turned to face outwards.

### Using the OmoLoc with the shoulder strap

For an additional stabilizing effect, the optional shoulder strap can be used if necessary. **10** Attach the shoulder strap firmly to the outside of the OmoLoc on the back. Put the orthosis on following the instructions described above.

**11** Guide the strap over the healthy shoulder to the front.

**12** Fasten the end of the shoulder strap with the square Velcro tab firmly to the forearm strap or **13** the abdominal strap. **14** Now undo the round Velcro end and adjust the length of the strap to fit.

### Taking off the orthosis

Undo the upper arm and forearm straps and the hand sling. Take your arm out of the straps. Undo the fastening with the finger pockets. Do up all the Velcro fastenings to avoid damaging your clothes or the OmoLoc.

## Cleaning instructions

**Note:** Never expose the product to direct heat (e.g. heaters, sunlight, in your car, etc.). This may cause damage to the material, which can impair the effectiveness of the product.

Please refer to the care instructions on the sewn-in label. This can be found on the upper edge of the abdominal strap:

1. Wash the product by hand.

2. Fasten the product’s strap

3. The product is not suitable for tumble drying.

Wash the product regularly.

To make a complaint, please contact the retailer that provided you with the product.

**Note:** Only clean goods are handled.

## Part of the body this product is used for

In accordance with the indications (shoulder). See intended use.

## Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

## Assembly and fitting instructions

The OmoLoc is supplied ready for use.

## Technical specifications / parameters, accessories

The OmoLoc is an orthosis for the shoulder joint. It consists of an abdominal strap, an upper arm strap, forearm strap, and hand strap, as well as an optional shoulder strap.

OmoLoc – shoulder strap, 129 7506 0000 000

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OmoLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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Polyoxymethylene (POM), Polypropylene (PP), Polyester (PES)

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