SISSEL® COLD THERAPY COMPRESSION

Gebrauchsanweisung
Operating manual
Mode d'emploi
Instrucciones

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With the **SISSEL® Cold Therapy Compression** you have purchased a universally applicable assistive device for joint problems or injuries.
A good decision, thank you.

The **SISSEL® Cold Therapy Compression** is an inflatable cold compress for knee/ elbow, wrist and ankle, which is shaped like a joint bandage. It can be applied to treat acute injuries (ligaments, joints) and swelling, and sudden or chronic joint aches or pains (for example rheumatic aches). It can also support treatments and therapeutic applications.

The combination of cold and pressure in shape of a bandage is very effective. Cold is known to reduce pain and swelling. Compression supports and stabilizes, increasing the effectiveness of the cold. Swelling for example, can be prevented more effectively or are clearly reduced. The freely removable inner Cold pad can be stored in the fridge or freezer.

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Combined Cold and Compression

Advantages at a glance:

- reliable reduction of pain and inflammation
- cold and compression provide comprehensive prevention of swelling
- additional joint stabilization of sports injuries, sprains etc.
- with pump for pressure regulation
- Slow Flow Gel provides long-lasting and safe temperature preservation
- convenient velcro closure, adjusts to joint perfectly
- hygienic and easy to disinfect
- can be applied on top of clothing in case of emergency
- available in 3 variations: wrist, knee/elbow, ankle

Indication:

- inflammations
- swellings
- pains
- sports injuries
- rheumatic conditions

Contraindication:

- Do not use in case of undefined results. Consult your doctor or therapist.
- Do not apply on open injuries or wounds (possibly cover first).

Material / Application

Material:

cover		Contents
<i>black</i>	<i>blue</i>	
100% nylon velours	65% Polyester 35% Cotton	Cellulose Aquadest

Care instructions: Clean with a moist towel and dry.

Desinfection: Spray or wipe disinfection. Spray, let soak, wipe if necessary. Use only suitable cleaners.

Temperature: Store compress at (max. – 18°) in your freezer or refrigerator to guarantee product is ready for use. The lower the temperature the longer the effectiveness of the cold reservoir.

Warning: Temperatures too low may cause frost damage to the skin! If applicable protect your skin with a towel.

Apply / Remove compress

▪ Apply

1. put on compress, close velcro closure (1)
2. open valve („ON“)
3. inflate (3)
4. adjust if desired. Press release valve (4)
5. close valve („OFF“) (2)
6. remove tube if desired (2)

Note:


Do not inflate compress too strongly, ensure proper positioning, do not put on too loosely.

Joint must be located at the joint opening (for knee and elbow compress)

▪ Remove

1. open valve („ON“) (2)
2. release air
 - press release valve or (4)
 - remove tube (5)
3. open velcro closures, remove compress, (1)
4. squeeze out remaining air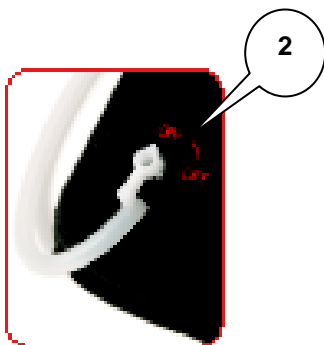

SISSEL® GmbH

Bruchstrasse 48

D-67098 Bad Dürkheim

Germany

www.sissel.com