THUASNE

ImmoVest

- fr** Gilet d'immobilisation d'épaule
en Shoulder immobilisation vest
de Bandage zur Ruhigstellung der Schulter
nl Schouderorthese voor immobilisatie
it Gilè di immobilizzazione della spalla
es Chaleco para inmovilización del hombro
pt Colete de imobilização do ombro
da Vest til skulderimmobilisering
fi Olkapään immobilisointiliivi
sv Väst för att immobilisera axeln
el Γλέκο ακινητοποίησης ώμου
cs Imobilizační ramenní závěs
pl Ortez do unieruchamienia barku
lv Pleca imobilizācijas veste
lt Peties imobilizavimo liemenė
et Õlaliigest fikseeriv ortoos
sl Steznik za imobilizacijo rame
sk Vesta na znehybnenie ramena
hu Vállrögzítő mellény
bg Жилетка за обездвижване на рамото
ro Orteză „tip vestă” de imobilizare a umărului
ru Жилет для обездвиживания плеча
hr Prsluk za imobilizaciju ramena
zh 肩部固定背心
ar صدرية لتثبيت الكتف

1	< 85 cm
2	86 - 95 cm
3	96 - 105 cm
4	> 105 cm

© Thuasne - 6167701 (2022-04)
© Studio Caterin

THUASNE
120, rue Marius AUFAN
92300 Levallois-Perret
France
www.thuasne.com

**UK
CA**

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SHOULDER IMMOBILISATION VEST

Description/Intended users

This device is a shoulder sling allowing the immobilisation of the shoulder and/or the elbow in internal rotation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyester – polyurethane – polyamide – cotton.

Properties/Mode of action

Shoulder immobilisation vest allowing for shoulder and/or elbow immobilisation in internal rotation.

Anatomical, wraparound design, covering the shoulder and elbow for reinforced protection.

Comfortable foam structure.

Removable strap for reinforced immobilisation.

Bilateral model.

Indications

Shoulder and/or elbow injuries or inflammatory conditions.

Conservative treatment for non-surgical proximal humeral fractures.

Shoulder dislocation.

Non surgical fractures of the clavícula and/or scapula.

Post-operative immobilisation (such as after rotator cuff surgery or dislocation reduction...).

Shoulder subluxation or dislocation prevention (joint instability).

Support for paralytic upper limb.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in certain post-operative conditions (multiple fractures, pin insertion, open fractures, etc.).

Do not use in the event of complex or surgical shoulder fractures, humeral fractures or rib cage fractures.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product when driving a vehicle.

Prolonged immobilisation can sometimes cause muscle weakness. If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

It is recommended to adequately tighten the device to achieve immobilisation without restricting blood circulation.

Wear the device above thin clothing.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

The help of another person is required to put on this product.

Preparing the device:

Unfold the device and open the arm sling ⓐ

Depending on the shoulder to immobilise, reverse the direction of the device: remove the arm sling ⓐ and place it on the opposite panel:

- Right shoulder/elbow to immobilize. ⓑ

- Left shoulder/elbow to immobilize. ⓒ

To facilitate application, fold the self-fastening closures of the sleeve, the abdominal and dorsal panels onto themselves.

The removable securing strap ⓐ is recommended for a better immobilisation.

Pre-position the removable securing strap ⓐ :

- one edge on the dorsal panel ⓐ on the indicated area ⓑ (see back view ⓑ)

- one edge on the top of the sleeve ⓐ (see front view ⓑ).

The strap will be adjusted when closing.

Fitting the device:

ⓑ Slide the arm needing immobilisation along the sleeve ⓑ.

ⓒ Place the shoulder pad ⓐ onto the shoulder to immobilised.

Unfold the self-fastening closure ⓐ of the dorsal panel ⓐ, attach it onto the front panel ⓐ. If needed, adjust the length of the abdominal panel ⓐ by folding it onto itself.

ⓑ If needed, adjust the closure strap ⓐ pre-positioned on the side of the immobilized shoulder.

ⓑ Place the arm needing immobilisation against the abdomen, fold and close the arm sling ⓐ onto the chest.

ⓒ Unfold the self-fastening closure and attach the end of the sleeve ⓐ under the arm sling ⓐ.

ⓑ ⓒ Reposition and tighten the removable securing strap ⓐ for a stronger immobilisation.

To remove the device, detach the removable securing strap, the sleeve, then the arm sling and detach the abdominal and dorsal panels.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Close the self-fastening tabs before washing. Machine washable at 30 °C (delicate programme). The device has been tested over 20 wash cycles. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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